

IOM ELECTRODES CARE





Preparation and application of electrode(s) should be supervised by a qualified medical professional.

Check the expiration date on the electrode packaging and verify that the sterilized packaging is intact. Follow local site procedures and discard any packages that have expired, or are not sealed and intact. Inspect electrodes for wear or damage prior to use. If electrode wire, termination, or surface area is worn, discard the electrode.

SGM is not responsible for injury, infection, or other damage resulting from improper storage, preparation, use, or disposal of electrodes.

IOM Electrodes

The labeling carries the following information:

LABEL - VISUAL	TITLE/MEANING/REFERENT	FUNCTION/DESCRIPTION
	DO NOT REUSE	Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.
	Use-by Date	Indicates the date after which the medical device is not to be used.
	Do Not Use if Package is Damaged	Indicates a medical device that should not be used if the package has been damaged or opened.
	Sterilized Using Ethylene Oxide	Indicates a medical device that has been sterilized using ethylene oxide.

Discard needle electrodes in a puncture-resistant sharps container following approved procedures for disposal of medical sharps.